Obsah balení

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2 plastové sáčky

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Absorbent pad

Marker pen

**Do not eat**, drink carbonated drinks, or perform oral hygiene for at least one hour before the collection of the sample.

The **powder** in the collection tube is needed for stabilization. Do not spill it out or consume it!

Dear customer, please note that the courier will wait a maximum of **10 minutes**.

**Max 10 min**

**1**

First of all, fill in the electronic application form at [lab-chromozoom.cz](https://lab-chromozoom.cz)

After successful registration, you will receive a **9-digit code** (in the format GHCXXXXXXXXX) by SMS / e-mail.

**2**

Put **5 ml** of water into the measuring cup.

**3**

Take a deep breath and cough.

**4**

Gargle the water for 20 – 30 seconds.

*If you accidentally swallow, repeat the procedure from the 2nd step.*

20 - 30 s

**5**

Spit the gargled contents back into the measuring cup.

**6**

Carefully pour the contents of the measuring cup into the collection tube.

*Do not spill out or consume the powder inside of the tube.*

**7**

Close the tube very carefully and shake it.

**8**

With a marker pen, fill in the name, phone number and 9-digit code from the electronic application form (in the format GHCXXXXXXXXX).

*The sample cannot be evaluated without the code!*

**9**

Stick the filled-out label onto the collection tube.

**10**

First place the tube, together with the absorbent pad, into the transparent bag, and then into the black zip bag.

**11**

Leave the packaged tube in a predesignated place for the courier. The courier will pick it up contactlessly.

You will **receive the results** by the next day via SMS and via e-mail.

*Do not put back in the box!*